InBody

DIAL H30

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Quick Start Guide

Quick Start Guide **USA**

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You can view and download the detailed guide on the InBody App and at <https://inbody.com/en>

1. Installing the InBody App

Search 'InBody' from Apple App store or Google Play store and install.

- Only mobile phones with iOS 13.0 or higher, Android 7.0 or higher, and Bluetooth 5.0 or higher can connect with the InBodyDial H30.
- Enable the Bluetooth function on your mobile device to establish a connection with the InBody App.
- You can record and manage your InBody Test results using the InBody App

android OS

iOS

2. InBody Test

1 Power On the Product and Measure Your Weight

1. Please turn on the product by stepping on the footplate (electrodes) with your bare feet and measure your weight.

*If you suspect there may be an error with the weight measured, step off the product and check whether there are objects or other foreign materials on the surface directly beneath the product, and the scale displays 0.0 kg. Then measure the weight again.

*You can also turn on the product by pressing the dial button and check if the scale displays 0.0 kg by pressing the dial button again.

2 InBody Test Posture

After the weight measurement is completed, please grip the handles and maintain them at the same height as your pelvis as shown in the picture and fully stretch your arms so that they don't touch your waist.



*An unregistered user can take the InBody Test by turning the dial button and inputting their height.

3 Check the InBody Test Results


If you turn or short-press the dial button, you can manually switch result outputs displayed.

*The results of changes compared to the last measurement, as well as the healthy ranges for those measurements, are displayed on the device's screen only when the InBody Test starts by utilizing the user recognition function or via the InBody App.


InBody Test Posture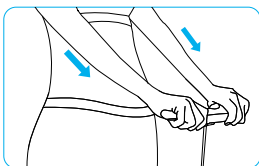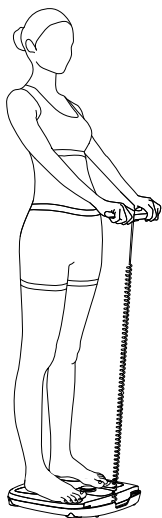

Please grip the handles and maintain them at the same height as your pelvis as shown on the left and fully stretch your arms so that they don't touch your waist.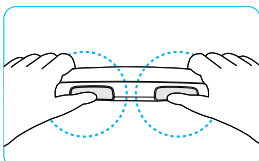

Place your thumb on the rectangular electrode.

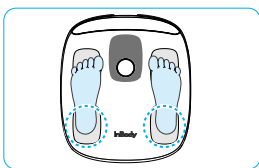Align your heel with the end of the foot electrode.

3. Cautions

- Persons wearing an implanted medical device (e.g., pacemaker) or a life-sustaining device (e.g., patient monitoring device) should not use the InBodyDial H30. InBody Co., Ltd. and its affiliates waive all liability, whether direct or indirect, stemming from any failure to abide by the warnings in this User's Manual.
- Bioelectrical Impedance Analysis (BIA) technology uses a micro electric current to perform the measurement, and is not harmful to the human body. However, please consult with your medical provider before using the product.
- Since InBodyDial H30 is not a medical device, it is not intended to diagnose, treat or analyze any illness.
- Anyone unable to properly assume the posture necessary to conduct an accurate test on their own should get help from others when taking a test. Likewise, it is recommended that children be assisted when conducting an InBody Test.
- For long-term storage, remove the batteries, pack the InBodyDial H30 in the original box and keep it on a flat surface.
- Dispose of the InBodyDial H30 and its batteries in compliance with local rules and regulations.

4. Product Warranty

- The warranty policy can be found in the User's Manual, which can be accessed at <https://inbody.com> or by registering your product on the InBody App, then clicking on  icon > Help > User's Manual.
- Warranty service will be effective only within the warranty period provided and only for an inherent manufacturer's defect.
- Any provisions not specified herein will be subject to the Fair Trade Commission notice of consumer dispute resolution standards.
- Only trained InBody personnel are allowed to repair and inspect InBodyDial H30. Please contact the Product Support team.

Contact

InBody Co., Ltd. [HEAD OFFICE]

InBody Bldg., 625, Eonju-ro, Gangnam-gu, Seoul, 06106 Korea

TEL: +82-2-501-3939 FAX: +82-2-578-2716

Website: inbody.com E-mail: info@inbody.com

InBody Europe [EUROPE]

Gyroscoopweg 122, 1042 AZ, Amsterdam, The Netherlands

TEL: +31-20-2386080 Website: nl.inbody.com E-mail: info.eu@inbody.com

InBody Oceania

U2/82-86 Minnie Street Southport, QLD 4215

TEL: +61 7 5681 1900 Website: au.inbody.com E-mail: support@au.inbody.com

Copyright

Reproduction, adaptation, or translation of this manual is prohibited without prior written consent from InBody Co., Ltd. under the copyright laws. This manual might have typographical errors, and its content can be changed without prior notice. InBody Co., Ltd. shall not be liable for any errors, incidental, or consequential damages that occurred by not complying with the content of the User's Manual. InBody Co., Ltd. reserves the right to modify the appearance, specifications, and other non-functionalities of the InBody Dial without prior notice to you.

InBody USA

You can view and download the detailed guide on the InBody App and at <https://inbodyusa.com>.

1. Installing the InBody App

Search 'InBody' from Apple App store or Google Play store and install.

- Only mobile phones with iOS 13.0 or higher, Android 7.0 or higher, and Bluetooth 5.0 or higher can connect with the InBodyDial H30.
- Enable the Bluetooth function on your mobile device to establish a connection with the InBody App.
- You can record and manage your InBody Test results using the InBody App

android OS

iOS

2. InBody Test

1 Power On the Product and Measure Your Weight

1. Please turn on the product by stepping on the footplate (electrodes) with your bare feet and measure your weight.

*If you suspect there may be an error with the weight measured, step off the product and check whether there are objects or other foreign materials on the surface directly beneath the product, and the scale displays 0.0 lb. Then measure the weight again.

*You can also turn on the product by pressing the dial button and check if the scale displays 0.0 lb by pressing the dial button again.

2 InBody Test Posture

After the weight measurement is completed, please grip the handles and maintain them at the same height as your pelvis as shown in the picture and fully stretch your arms so that they don't touch your waist.



*An unregistered user can take the InBody Test by turning the dial button and inputting their height.

3 Check the InBody Test Results


If you turn or short-press the dial button, you can manually switch result outputs displayed.

*The results of changes compared to the last measurement, as well as the healthy ranges for those measurements, are displayed on the device's screen only when the InBody Test starts by utilizing the user recognition function or via the InBody App.


InBody Test Posture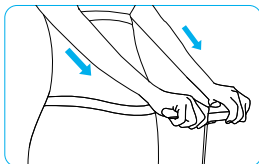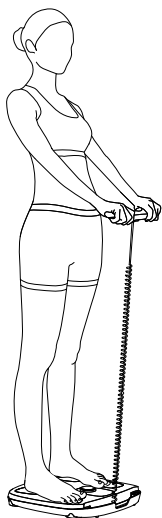

Please grip the handles and maintain them at the same height as your pelvis as shown on the left and fully stretch your arms so that they don't touch your waist.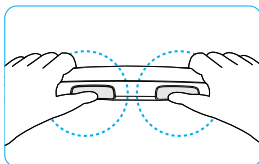

Place your thumb on the rectangular electrode.

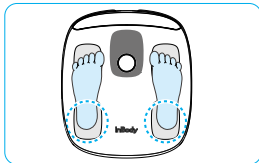Align your heel with the end of the foot electrode.

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4. Product Warranty

- The warranty policy can be found in the User's Manual, which can be accessed at <https://inbodyusa.com> or by registering your product on the InBody App, then clicking on  icon > Help > User's Manual.
- Warranty service will be effective only within the warranty period provided and only for an inherent manufacturer's defect.
- Any provisions not specified herein will be subject to the Fair Trade Commission notice of consumer dispute resolution standards.
- Only trained InBody personnel are allowed to repair and inspect InBodyDial H30. Please contact the Product Support team at Support@inbody.com.

Contact

InBody USA [USA]
 13850 Cerritos Corporate Dr. Unit C Cerritos, CA 90703 USA
 TEL: +1-323-932-6503 FAX: +1-323-952-5009
 Website: inbody.com E-mail: info.us@inbody.com

Copyright

Reproduction, adaptation, or translation of this manual is prohibited without prior written consent from InBody Co., Ltd. under the copyright laws. This manual might have typographical errors, and its content can be changed without prior notice. InBody Co., Ltd. shall not be liable for any errors, incidental, or consequential damages that occurred by not complying with the content of the User's Manual. InBody Co., Ltd. reserves the right to modify the appearance, specifications, and other non-functionalities of the InBody Dial without prior notice to you.

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